



 Instruction Manual

assembly instructions





Natriflow technology uses two pumping patterns to simulate a baby's nursing rhythm. During the "Flow Start" mode, the pump mimics a breastfeeding baby's initial rapid nursing action that stimulates the milk ejection reflex or "start of the milk flow". Once milk begins to flow, Mother switches to the second mode, called "Expression," which mimics the infant's longer feeding cycle in which slower, deeper suckling helps to maximize milk flow in less time.



flow start

Increase the suction pressure and press the release valve (centre button) sporadically every 2-3 seconds to mimic babies rapid nursing action.



expression

Decrease the suction pressure and deactivate the release valve for continuous suction mimicing babies longer expression.



Clean



Clean



Wash pump and accesories



Rinse pump and accesories



Sterilise

or




Dishwasher



Do not microwave!

technical specifications

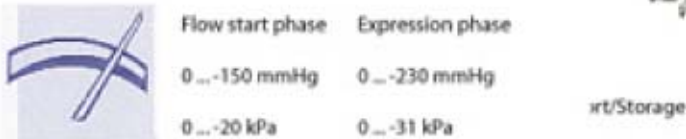

The Natripump Adaptor Shield makes it easier for Mothers to draw breast milk for baby. The silicone adaptor shield with bubble design simulates baby's suction. The adaptor shield expands to the rim of the suction cup for massaging mother's breast during expression. This revolutionary technology enables breast milk to be expressed as a natural flow.

storing breast milk



Store only in climate controlled area

other cherub products...



Natriflow Electric Breast Pump

The Cherub Electric Pump utilises the Natriflow™ technology through a "stop/start" PUSH button that simulates babies sporadic suction combined with an adjustable rotating dial to regulate the suction level, and to maintain a natural flow of breast milk. The only Electric Breast Pump to use Natriflow and Natriflow technology.



Cherub Aqua Steriliser

With a fast 8 minute sterilising cycle the Aqua 6 bottle steriliser is all you need to ensure your babies products are sterilised effectively. Comes complete with dishwasher safe basket, jug and tongs.



Cherub Nightvision Monitor

This fully wireless colour LCD monitor comes complete with nightvision and sound to ensure you can see and hear baby all the time. Connect the monitor to your tv and view and record baby on the big screen! Comes complete with rechargeable batteries, monitor belt clip, stand and tv cord.



3-5 days



6 months




12 months



thaw in warm water



do not thaw in microwave

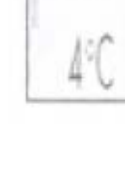

after thawing store at low temperature



Follow these instructions unless otherwise advised by your lactation consultant or physician